

**MED** **STOCK**<sup>®</sup>

# FABRIC ISLAND DRESSING

**HIGHLY PERMEABLE WITH GOOD  
ADHESIVE STRENGTH**

Medstock Fabric Island Dressing is made of a spunlace, non-woven fabric coating with a specialised medical-grade acrylic adhesive and an absorbent fabric pad.

AVAILABLE IN 5 SIZES

## Key Functions and Features


- + Creates an occlusive environment, whilst being absorbent and permeable to oxygen and moisture
- + Hypoallergenic
- + Mild peeling strength
- + Does not roll up or fall off skin
- + Prevents bacterial invasion and accelerates wound healing

## General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Can be left in place for up to 5 days depending on exudate level and type of wound
- + Can be used as a secondary dressing over hydrocolloid paste, alginate and other primary dressings

## Indications for Use

- + Suitable for wounds with nil to mild exudate
- + Grazes, abrasions and superficial burns
- + Surgical wounds

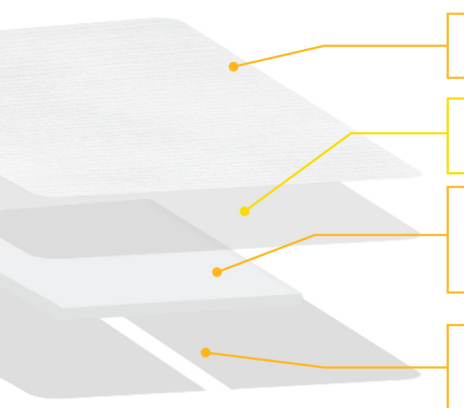

- First Layer:** Spunlace non-woven
- Second Layer:** Medical grade PSA
- Third Layer:** Cotton island pad absorption rate is  $\geq 5$  times the original pad weight
- Backing Release Paper:** Glassine release liner



## Contraindications / Precautions

- + Seek medical advice immediately for all serious wounds and burns
- + If reddening or sensitisation occurs, discontinue use and consult a physician

## Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

## Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

## Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

### Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS100507NSI	5cm x 7cm	100 pcs/box
MS100710NSI	7cm x 10cm	100 pcs/box
MS1001010NSI	10cm x 10cm	100 pcs/box
MS100815NSI	8cm x 15cm	100 pcs/box
MS501020NSI	10cm x 20cm	50 pcs/box