

MEDSTOCK[®]

**FOAM
NON-
ADHESIVE
DRESSING**

**A FLEXIBLE AND COMFORTABLE
FOAM DRESSING THAT PROVIDES
RAPID EXUDATE ABSORPTION**

Medstock Foam Non-Adhesive is a highly absorbent polyurethane foam dressing that contains carboxymethylcellulose (CMC) hydrophilic granules, promoting and maintaining a balanced, moist healing environment.


AVAILABLE IN 2 SIZES

Key Functions and Features

- + High absorbency and fluid handling capacity
- + Reduces the risk of maceration to wound and periwound
- + Embedded with carboxymethylcellulose hydrophilic granules that promotes a moist wound healing environment
- + Ability to vertically wick exudate away from the wound and retain exudate in the core of the foam
- + Non-adherent to the wound preventing damage to healing tissue
- + Surface layer consists of a semi-permeable polyurethane film to protect the wound against water, bacteria and foreign matters
- + Good elasticity and flexibility which minimises the risk of pressure on the wound bed
- + Effective odour control for malodorous wounds

Indications for Use:

- + Suitable for wounds with low to high exudate
- + Adaptable to different phases of wound healing
- + Partial thickness burns
- + Pressure ulcers
- + Venous ulcers
- + Arterial ulcers
- + Mixed aetiology ulcers

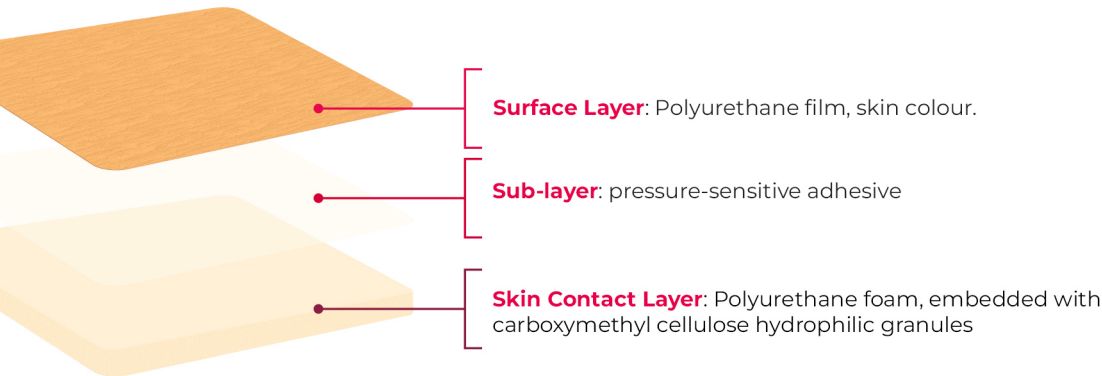

General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Can be left in place for up to 7 days but should be changed immediately if 3/4 strikethrough (visible exudate) is apparent
- + Can be used as a secondary dressing over hydrocolloid paste, alginate and other primary dressings
- + If necessary, the dressing can be cut to the required shape and size
- + Additional tape or other means of fixation must be used to secure dressing



Contraindications / Precautions

- + If reddening or sensitisation occurs, discontinue use and consult a physician

Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised by radiation.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS0505F	5cm x 5cm	10pcs/box
MS1010F	10cm x 10cm	10pcs/box