

MED **STOCK**[®]

FIRST AID

**NON-STICK
PAD**

DRESSING

**ABSORBS UP TO 8 TIMES
ITS OWN WEIGHT**

Medstock Non-Stick Pad is an absorbent, non-adhesive dressing. It is made of a heat-laminated, porous film skin-contact layer and absorbent pad.

AVAILABLE IN 2 SIZES

Key Functions and Features

- + Soft, conformable, and comfortable to wear
- + Can be cut to the required shape and size of the wound
- + Provides protection for fragile tissue with its low-adherent properties




General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Can be left in place for up to 5 days depending on exudate level and type of wound
- + Can be used as a secondary dressing over hydrocolloid paste, alginate and other primary dressings
- + If necessary, the dressing can be cut to the required shape and size
- + Additional tape or other means of fixation is necessary to secure dressing

Indications for Use

- + Suitable for wounds with nil to mild exudate
- + Grazes, abrasions and superficial burns
- + Surgical wounds

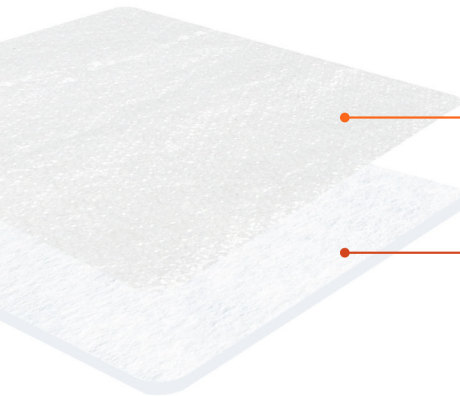

First layer: Heat laminated porous film

Second Layer: Absorbent pad made from 70% viscose and 30% polyester



Contraindications / Precautions

- + Seek medical advice immediately for all serious wounds and burns
- + If reddening or sensitisation occurs, discontinue use and consult a physician

Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

| Product Code | Size | Pcs/Box |
|--------------|-------------|-------------|
| PHA050NSP | 5cm x 5cm | 50 pcs/box |
| PHA1010NSP | 10cm x 10cm | 100 pcs/box |