

MEDSTOCK[®]

WOUND CLOSURE DRESSING

EFFECTIVE IN PREVENTING SCAR FORMATION

Medstock Wound Closures are non-invasive, adhesive skin closure devices that are designed to provide comfortable support that facilitates quick wound closure.

Key Functions and Features



- + Provides wound support and facilitates wound closure
- + Non-invasive technology can help reduce risk of infection and scar formation
- + Reduces need for wound sutures to facilitate wound closure

General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Can be left in place for up to 5 days or once healing is advanced enough to ensure the wound is closed
- + Medstock Wound Closure should be spaced approximately 3mm apart
- + Do not apply wound closure under high tension as this may lead to impaired healing
- + Consider secondary dressing to secure and cover
- + On removal, pull slowly and gently whilst supporting skin

Indications for Use

- + Closure of lacerations and surgical incisions
- + Can be used in conjunction with sutures/staples
- + Supports wound after sutures/staples are removed

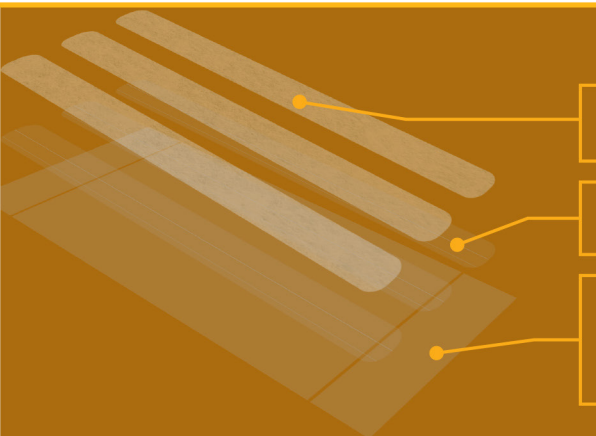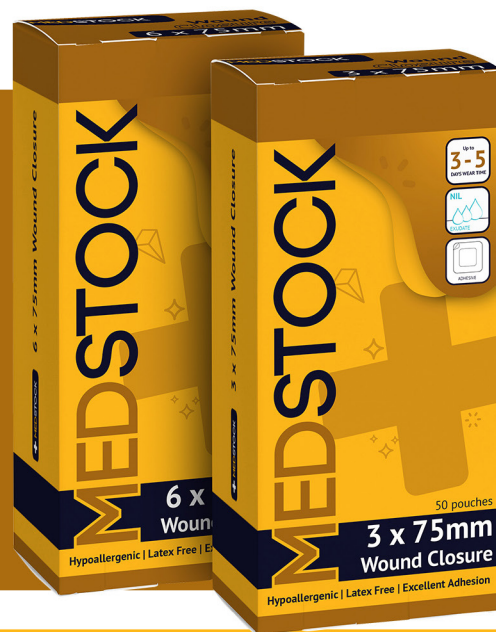

First Layer: Non-Woven Fabric

Second Layer: Acrylic adhesive

Backing Release Paper:
CCK release paper



Contraindications / Precautions

- + Seek advice from a qualified healthcare professional prior to application
- + If reddening or sensitisation occurs, discontinue use and consult a physician
- + Wound closures are not intended for use on infected wounds

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS3WC	3mm x 75mm	50 pouches/box
MS6WC	6mm x 75mm	50 pouches/box