

MEDSTOCK™ QUICK-START DRESSING GUIDE

Simplified wound care solutions for community pharmacy

At Medstock, we believe advanced healing should be accessible to everyone. This guide is designed to help your team navigate our professional-grade range in seconds. By aligning common wound presentations with our signature Colour-Coding System, you can provide fast, accurate, and expert advice to every customer, ensuring they leave with the right dressing to support optimal recovery.

BASIC & ADVANCED WOUND CARE

Go Modular

Magnetic display units designed for all spaces and range requirements - provided free into stores.



Grazes and abrasions

Wound appearance: Superficial skin loss, minor bleeding, little to no fluid.

Fabric island dressings



First-line protection for minor wounds.

Thin hydrocolloid dressings



Moisture donation for dry wounds.

Non-stick pads, fixer roll



An absorbent pad that provides a low risk of adherence to wound tissue.

Secondary adhesive fabric fixation for securing pads and dressings.

Disclaimer: This is a general guide for informational purposes only. This is not intended to replace medical advice. Always refer customers to seek medical advice, especially if the wound is worsening, deep or covers a large surface area, appears infected or is slow to heal.

MEDSTOCK™ QUICK-START DRESSING GUIDE

Simplified wound care solutions for community pharmacy

Minor cuts and lacerations


Wound appearance: Small, shallow linear breaks in the skin, usually less than 5cm, in body areas that are not prone to movement & flexion. Mild bleeding only.

Wound closures



Secure and support wound edges to encourage healing.

Non-stick pads, fixer roll

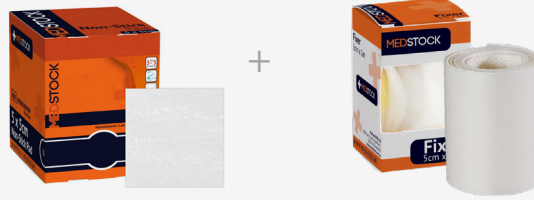

An absorbent pad that provides a low risk of adherence to wound tissue.

Secondary adhesive fabric fixation for securing pads and dressings.

Skin tears and fragile skin

Wound appearance: Fragile, thin, or aged skin with poor elasticity - or for gentle and comfort needs. May be dry or weeping.

Silicone Foam



A 5 layered foam dressing that is highly absorbent and gentle on fragile wound tissue and its surrounding skin.

Silicone Foam Lite



A 4 layered foam dressing that is designed for lightly weeping wounds only, yet gentle on fragile wound tissue and surrounding skin.

Silicone Wound Contact Layers



Wound interface dressing that is designed to prevent wounds sticking to secondary dressings. Placing this layer between the wound and outer dressing for a gentle, atraumatic layer to protect healing tissue.

Disclaimer: This is a general guide for informational purposes only. This is not intended to replace medical advice. Always refer customers to seek medical advice, especially if the wound is worsening, deep or covers a large surface area, appears infected or is slow to heal.

MEDSTOCK™ QUICK-START DRESSING GUIDE

Simplified wound care solutions for community pharmacy

Surgical wounds

Wound appearance: Closed incision, post-operative wound, little to no fluid.

Transparent island dressings



Breathable, absorbent waterproof protection.

Transparent film



Breathable and waterproof protection with full visibility of wound.

Fabric island dressings



Cost-effective protective dressing with an absorbent pad.

Transparent foam island dressings



Breathable, waterproof absorption that incorporates a foam for more moderate fluid leakage. It contains hydrating granules that form a hydrating gel that promotes wound healing.

Wound Closure



Non-invasive, adhesive skin closure devices that are designed to provide comfortable support that facilitates quick wound closure.

Disclaimer: This is a general guide for informational purposes only. This is not intended to replace medical advice. Always refer customers to seek medical advice, especially if the wound is worsening, deep or covers a large surface area, appears infected or is slow to heal.

MEDSTOCK™ QUICK-START DRESSING GUIDE

Simplified wound care solutions for community pharmacy

Minor burns

Wound appearance: Shallow burns with broken or blistered skin, Low fluid.

Thin hydrocolloid dressings



Maintains moisture balance for minor burns. Can place dressing over a hydrogel for extra moisture.

Non-stick pads, fixer roll



An absorbent pad that provides a low risk of adherence to wound tissue.

Secondary adhesive fabric fixation for securing pads and dressings.

Ulcers

Wound appearance: Long-standing open wounds with variable fluid.

Foam non-adhesive dressings



Ongoing absorption for ulcerated wounds.

Alginate dressings



Higher absorption and fluid control. Pair with absorbent fixation dressing.

Super absorber dressings



For wounds that are very leaky and not bleeding, and require regular dressing changes. Can be used under compression garments (up to 40 mmHg).

Disclaimer: This is a general guide for informational purposes only. This is not intended to replace medical advice. Always refer customers to seek medical advice, especially if the wound is worsening, deep or covers a large surface area, appears infected or is slow to heal.

MEDSTOCK™ QUICK-START DRESSING GUIDE

Simplified wound care solutions for community pharmacy

Highly exuding or bleeding wounds

Wound appearance: Open wounds with high fluid output or active bleeding.

Alginate dressings



Absorption and minor bleeding control. Requires secondary fixation dressing to secure.

Wound appearance: Extra large amounts of exudate and traumatic wounds.

Super absorber dressings



For wounds that are very leaky and not bleeding; and require regular dressing changes. Can be used under compression garments (up to 40mmHg)

General advice for safety:

- Scan the QR code on the product pouch for further information on application, indications for use and advice on safe use.
- Always refer to the instructions prior to use.

Disclaimer: This is a general guide for informational purposes only. This is not intended to replace medical advice. Always refer customers to seek medical advice, especially if the wound is worsening, deep or covers a large surface area, appears infected or is slow to heal.